

CLEANING INSTRUCTIONS FOR:

- **Polyurethane**
- **Polycarbonate**
- **Lectrolite®**

Clean with a non- aggressive, disinfectant cleaner. Casco urethane covers were tested with the following chemicals. None were found to cause fading or damage to the material.

NOTE:

- **It is necessary that the disinfectant manufacturer's dilution ratios and application procedure be followed.**
- **After the disinfectant manufacturer's dwell time has been achieved, the surface must be immediately and thoroughly rinsed with warm water.**
- **Do not allow the disinfectant to pool on the surface longer than the disinfectant manufacturer's dwell time.**
- **Do not allow the disinfectant to dry on the surface.**

- Advanced Sterilization Products: Cidex
- Airkem: A456-N
- Caltech: Dispatch
- Cosco: Tincture of Green Soap
- Ecolab: A456—II Disinfectant Cleaner
- Isopropyl Rubbing Alcohol 70%
- METREX: Cavi-Wipes
- METREX: Cavicide
- PDI: Super Sani-Cloth Plus Germicidal Cloth
- Spartan: SD-20
- Spartan: Hepacide Quat II
- Veridien: Viraguard Wipes
- VIROX: Accel
- VIROX: 256
- Clorox: Healthcare Bleach Wipes
- Sporicidin: Sterilization Solution
- Steris: Coverage Wipes Plus